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## Instructions for Use

# Dentamed P&B II

Light cure one component bonding agent  
for the total-etch-technique

Dentamed P&B II is a simple to use light cure one component bonding agent. It is designed for strong bonding of light cure composites and composites to etched enamel and dentine and to nonprecious and precious metals.

Its strong adhesion to etched enamel or dentine proceeds on principles similar to that occurring with glassionomer cements. Good, long-lasting adhesive strength and good biocompatibility are attained by polycarboxylic acids.

Dentamed P&B II is compatible with all current brands of visible light cure composite materials. It is ethanol-based and hydrophilic. Dentamed P&B II can be used on slightly moist dentin surfaces (wet bonding technique).

### Indications

Bonding for light cure composites.

### Contraindications

The placement of Dentamed P&B II is contraindicated if a dry working area or the recommended application technique are not possible. Also do not use Dentamed P&B II if the patient is known to be allergic to any of the ingredients.

### Side effects

In singular cases, Dentamed P&B II may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

### Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

### Application

#### 1. Isolation

Rubber dam is the recommended method of isolation.

#### 2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (**0.5 - 1.0 mm**) bevel placed in the enamel to increase the surface area for greater bond strength.

#### 3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of calciumhydroxide material.

#### 4. Enamel and Dentine Conditioning

Recommended is the total etch technique. Apply **Dentamed Etching Gel** onto the enamel and dentine surfaces beginning with the enamel bevels. Leave the etching gel in place for **20 seconds**. On primary teeth and teeth high in fluoride, a **60 second** etch is recommended. Rinse for 20 seconds with water. Dry it in a water and oil-free airstream, but **do not** desiccate. A slightly wet dentine surface is important for the function of Dentamed P&B II. The etched enamel bevel should have a chalky white appearance.

#### Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

#### 5. Application of Dentamed P&B II

Apply Dentamed P&B II generously with a brush onto the enamel and dentine surfaces for **30 seconds** with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously

with oil free air for about 15 seconds to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Cure the Dentamed P&B II coating by exposing its entire area to a dental LED light unit for **20 seconds** before application of a second layer of Dentamed P&B II.

#### Notes:

Do not rinse off the Dentamed P&B II! If not used immediately, place dispensed Dentamed P&B II in subdued light to prevent premature polymerization by incident light. The Dentamed P&B II will **not** self cure.

#### 6. Application of a Second Layer of Dentamed P&B II

Apply again the Dentamed P&B II generously with a brush onto the adhesive surfaces as described under **point 5** before placement of a light cure composite.

#### 7. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

#### Storage

Do not store above 25°C (78°F). Protect from direct sunlight. Do not use after expiry date.

#### Warnings

- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized Dentamed P&B II may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

#### Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

#### Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!**  
**For dental use only!**

Vertrieb:

**Dr. Ihde** Dental GmbH  
Erfurter Strasse 19  
D-85386 Eching  
Tel. +49 89 319 761-0  
info@ihde-dental.de

Manufacturer:

P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany