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Dentamed E&B2

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Manufacturer:
P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany

Instructions for Use**Dentamed E&B2****Self-etching dual cure adhesive**

Dentamed E&B2 is a simple to use self-etching dual cure adhesive for a durable and strong bonding of composites, compomers and resin modified glass ionomer cements to enamel and dentin. Dentamed E&B2 works optimally under slightly wet conditions.

Dentamed E&B2 consists of the components PART A and PART B that were mixed before application. It is bonding to dual cure, self cure and light cure composite restorative materials.

Indications

Adhesive for:

- Core build-ups and cementing of posts with dual and self cure composites
- Cementing of posts with dual and self cure composite cements
- Cementing of inlays, onlays, crowns and bridges with dual and self cure composite cements
- Composite based restorations with dual cure, self cure and light cure composites

Side effects

In singular cases, Dentamed E&B2 may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of Dentamed E&B2. Neither store the material in proximity of eugenol containing products, nor let the material allow coming into contact with materials containing eugenol.

Application**1. Isolation**

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of calcium hydroxide material.

4. Application of Dentamed E&B2

One drop of Dentamed E&B2 PART A and one drop of Dentamed E&B2 PART B were combined in a mixing pallet and mixed for **5-10 seconds**.

Notes:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

4.1. Application for Core Build-ups, Cementing of Inlays, Onlays, Crowns and Bridges and for Composite based Restorations

Apply the homogeneous mixture generously with a brush onto the slightly wet enamel- and dentin surfaces (as they are usual in clinical procedures) for **30 seconds** with agitation. The material should build a homogeneous layer. Air thin for **10 seconds** to remove the volatile components and to disperse the adhesive. Then light cure for **20 seconds** with a dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) and place the restorative material. **For a maximal adhesion it is strongly recommended to use the light cure mode.**

If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application of the adhesive air thin to remove all volatile components. There must remain a sticky layer. Then apply the dual cure, self cure or light cure restorative material directly.

4.2. Application in the Root Canal (Cementing of Posts)

Prepare and clean the root canal with e.g. sodium hypochlorite solution, rinse and remove excess solution from the canal with a soft paper point.

Apply the homogeneous mixture of Dentamed E&B2 Part A and Dentamed E&B2 Part B generously with a brush onto the slightly wet root canal walls for **15 seconds** with agitation. Repeat procedure **1 – 2 times**. All the dentin surfaces must kept wet with the primer over the specified time.

Remove excess Dentamed E&B2 with dry paper points. Dry the root canal cautiously with oilfree air for **15 seconds** to remove all volatile components and to disperse the adhesive to an even layer. Light cure all areas that are available for a dental curing unit for **20 seconds**.

5. Restorative Placement, Cure and Finishing

Refer to manufacturer instructions for placement of the dual cure, self cure or light cure restorative material.

Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage

Do not store above 25 °C (77 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!
For dental use only!**

Vertrieb:

Dr. Ihde Dental GmbH
Erfurter Strasse 19
D-85386 Eching
Tel. +49 89 319 761-0
info@ihde-dental.de

Manufacturer:
P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany