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Manufacturer:

P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany

**Instructions for Use****Dentamed Etching Gel****Thixotropic gel containing 37% phosphoric acid**

Dentamed Etching Gel is a 37% phosphoric acid gel with an excellent thixotropy.

From the 2 ml syringes with very thin disposable application cannulas it can be applied easily and safely on only that areas that are really desired to be etched.

**Warnings**

Dentamed Etching Gel contains phosphoric acid and is corrosive. Avoid contact with eyes, mucous membrane and skin (safety glasses are recommended for both patients and dentists). After contact with eyes immediately rinse with copious amounts of water for at least 15 minutes and consult a physician/ophthalmologist. After skin contact immediately rinse with soap and water.

**Preparation of Dentamed Etching Gel syringe**

Screw off the screw cap and substitute by the application needle. **Store the screw cap!**

**Note:**

The application needle might be bend in a slight arc, if desired, to facilitate application.

**Handling precaution:**

Squeeze out the first small drop on a pad to ensure proper handling and prevent "shooting" out of the etching gel by entrapped air.

**Application of Dentamed Etching Gel**

Apply **Dentamed Etching Gel** onto the enamel and dentine surfaces beginning with the enamel bevels. Leave the etching gel in place for **20 seconds**. On primary teeth and teeth high in fluoride, a **60 second** etch is recommended. Rinse for 20 seconds with water. Dry it in a water and oil-free airstream, but **do not** desiccate. A slightly wet dentine surface is important for the function of the used priming and bonding systems. The etched enamel bevel should have a chalky white appearance.

**Etching precaution:**

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

After use replace disposable application cannula by the screw cap. Discard used application cannula.

**Application of Adhesive**

Apply the adhesive of your choice ( e.g. Dentamed P&B II) using a brush on the etched saliva-free enamel and dentine surfaces according to the instructions of the manufacturer.

**Restorative placement, cure and finishing**

Refer to manufacturers instructions for placement, curing and finishing of restorative materials.

**Storage**

Do not store above 25 °C (78 °F). Do not use after expiry date.

**Warranty**

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

**Limitation of Liability**

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!  
For dental use only!**

Vertrieb:

**Dr. Ihde** Dental GmbH  
Erfurter Strasse 19  
D-85386 Eching  
Tel. +49 89 319 761-0  
info@ihde-dental.de

Manufacturer:

P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany