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Manufacturer:
P.L. Superior Dental Materials GmbH, 22607 Hamburg, Germany

Enamelite

Light Cure Easy Flowable Microhybrid Composite

Enamelite is a visible light cure easy flowing microhybrid composite for anterior and limited posterior restorations and for sealing. It is based on methacrylate-resin and inorganic filler particles of 0.05-1 µm. The total filler load is 61% and the total filler volume 41%. The composite material meets the requirements of ISO 4049.

Indications

- Anterior restorations (class III, IV, V)
- Small class I and II restorations
- Lining of cavities
- Sealings

Side Effects

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calciumhydroxide material.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Preparation of Tooth and Color Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate Enamelite shade.

Enamelite is shaded according to VITA®-Shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.

Best pulp protection in case of deep excavations is the photo-activated disinfection (PAD). If this is not possible, deep excavations should be covered with a thin layer of calciumhydroxide material.

Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with Dentamed Etching Gel and subsequent application of a light cure bonding agent (e.g. Dentamed P&B II) is recommended. Otherwise use a self etching bonding agent (e.g. Dentamed E&B) before application of Enamelite.

Placement of Enamelite

Apply Enamelite in the selected shade. Place it by using a suitable instrument. Transparent matrix strips may be used.

Curing of Enamelite

For an optimum result apply **Enamelite** in **layers of max. 1 mm**. Light cure each increment separately.

By using a polymerization unit with a **light intensity of at least 1000mW/cm²** and a **wavelength range of 400 to 500 nm** (blue light) cure each increment as follows:

Lighter shades (e.g. A1, A2, B2) **20 seconds**

Darker shades (e.g. A3.5) **30 seconds**

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

Additional Notes

- Do not use any resin to adjust viscosity of composite restorative material.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!

Distributor:

Dr. Ihde Dental GmbH
Erfurter Strasse 19
D-85386 Eching
Tel. +49 89 319 761-0
info@ihde-dental.de