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Type of preparation

PLIWA® Lemon Fresh AF is a ready-to-use high-performance preparation developed using the latest technologies for the rapid disinfection of medical products and medical equipment which acts on an ethanol and quats basis as described in the MPG (German Medical Devices Act). PLIWA® Lemon Fresh AF produces a pleasant, fresh scent and is easy on the skin. PLIWA® Lemon Fresh AF fulfils the legal occupational safety requirements laid down in TRGS 525/540.

Areas of use

PLIWA® Lemon Fresh AF is a surface disinfectant suitable for use in areas where rapid disinfection is required:

- In medical and dental practices
- Rapid disinfection of operating inventory and tables between operations
- Bed frames, mattresses, covers and pillows
- Bathtubs and toilet seats
- Clothing such as tunics, OP shoes, rubber aprons, etc.
Not to be used on surfaces sensitive to alcohol, such as acrylic glass!

Application

PLIWA® Lemon Fresh AF can be used as both a wipe-on disinfectant and as a disinfectant spray. With wipe-on disinfection (as per the recommendation of the RKI), moisten a disposable cloth with a sufficient amount of PLIWA Lemon Fresh AF and then use this to wipe the surfaces. The surface must be made completely moist (by wet wiping) during the necessary soaking time. The spray disinfectant process is only recommended for surfaces that are difficult to access. PLIWA® Lemon Fresh AF is a ready-to-use solution. With the spray disinfectant process, apply it undiluted to objects and surfaces and allow it to take effect. Ensure complete moistening, and if necessary remove excess amounts with an uncoloured, disposable cloth after the soaking time.

Application concentrations / application time**Application time as defined by Association of**

Applied Hygiene:			
Bactericide: (high load)		5 minutes	
MRSA, S. aureus, E. hirae,		30 seconds	
P. aeruginosa, E. coli,			
C. albicans, P. mirabillis			
Levurocide: (high load)	DIN EN 13624	30 seconds	
Candida albicans			
Acinetobacter baumannii	DIN EN 13727	30 seconds	
Tbc, Mycobacterium terrae	DIN EN 14348	1 minute	

Full virucide**enveloped viruses**

BVDV (HBV/HIV/HCV) (Influenzavirus,		30 seconds	
H1N1/H5N1, Herpes-, Corona-, SARS- CoV)			
Vaccinia Elstree strain		30 seconds	

unenveloped viruses

Adenovirus Typ 5 (low load)	DIN EN 14476	5 minutes	
Human rotavirus (low load)	DIN EN 14476	1 minute	
Poliovirus type 1 (low load)	DIN EN 14476	5 minutes	
MNV Norovirus (high load)	DIN EN 14476	5 minutes	

Product composition

The ready-to-use solution contains: Ethanol, 2-Propanol, N.N-Didecyl-N,N-dimethylammonium chloride, fragrance, pH regulators

Chemical and physical details

Appearance: clear fluid
pH: approx. 5,0 - 6,0
Density (20° C): 0,900–0,920
Flashpoint (EN 22719) 22,75 + 0,5 ° C

Precautionary statements

H226	Flammable liquid and vapour
H319	Causes serious eye irritation
P210	Keep away from heat / sparks / open flames / hot surfaces. – No smoking.
P243	Take precautionary measures against static discharge.
P280	Wear protective gloves / protective clothing / eye protection / face protection.
P303 / P361 / P353	IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.
P305 / P351 / P338	IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Personal Protection:

Wear protective equipment (gloves, goggles, mask, protective clothing). Keep unprotected persons away. Ensure sufficient ventilation. Avoid contact with the eyes. Do not inhale vapours / aerosol fumes. Keep sources of ignition away. Between 15 ml and 20 ml of the preparation is required to disinfect 1 m².

Environmental information

L.C. PLIWA® manufactures its products in accordance with advanced, safe and environmentally-friendly procedures, cost-effectively and in accordance with high quality standards (EN ISO 9001, EN ISO 13485). Biological degradability, which has been verified in expert reports, shows that disposal has a very low impact on residual water and the environment and is thus environmentally friendly. Protect from fire

Labelling

CE 0124 Class IIa / MD 0108
UN 1987
baua Reg.-Nr. N-57231

Storage instructions

Do not use after expiry date. Store in a cool place. Heating can cause pressure increases and may lead to bursting. Protect against heat and direct sunlight. Store container in a well-ventilated place.

Special note

Always read the Instructions for Use and the Material Safety Data Sheet before use.

Keep out of reach of children.**Delivery units**

5 L REF 10-412

Expert opinions: Available on request.

Revision date 2016-05

L.C.Pliwa®

Pliwastr. 2
DE-34323 Malsfeld-Ostheim
Tel. +49 (0)5661 7317 0
Fax +49 (0)5661 7371 10
info@pliwa.de
www.pliwa.de

Distribution by

Dr. Ihde Dental AG
Dorfplatz 11
8737 Gommiswald
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Warning

CE 0124

Legend

	Read instructions
	Expiration date
	LOT Charge number

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