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CE 0029

IF IN EYES: Rinse cautiously with water for several minutes. Remove contactlenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep cool. Dispose of contents / container in compliance with local / regional / national / international regulations.

Type of preparation

IHDENT® Dentatrend® Disinfection wipes are disinfection wipes for cleaning small areas, equipment, hand pieces and contra-angle hand pieces, and so on. The disinfection wipes are saturated in IHDENT® Dentatrend® Spray disinfection, which acts on an ethanol and quats basis, as described in the MPG (Medical Devices Act). IHDENT® Dentatrend® Disinfection wipes are aldehyde-free and give off a fresh perfume. Their benefits include broad microbiological efficacy against bacteria, fungi and viruses, good compatibility with materials, and the ability to dry quickly without leaving marks. IHDENT® Dentatrend® Disinfection wipes fulfil the legal occupational safety requirements laid down in TRGS 525/540.

Areas of use

IHDENT® Dentatrend® Disinfection wipes are suitable for use as quick disinfectant wipes as defined in the Medical Devices Act, in areas involving non-invasive medical products:

- Medical and dental practice
- Rapid disinfection of operating inventory and tables
- Bed frames, rubberised mattresses
- Bathtubs and toilet seats
- Clothing such as OP shoes, rubber aprons
- Treatment chairs and the surfaces of treatments units
- Surfaces in ambulances, supply trolleys, etc.

Not to be used on surfaces sensitive to alcohol, such as acrylic glass!

Application

Cut open one of the top corners of the IHDENT® Dentatrend® Disinfection wipes refill bag. Place the refill bag (fleece roll and saturation solution) into the dispensing tub, then pull the first wipe out from the middle of the roll, thread it through the opening in the dispensing tub lid, and close the tub. Wipe down objects, surfaces and parts well, and allow to dry. Do not use disinfected areas, objects or parts until the specified application time has lapsed.

Microbiology

IHDENT® Dentatrend® Disinfection wipes are effective against: Bacteria (incl. TB, Mycobacterium terrae, low load) / Levurocide (Candida albicans) / Viruses (HBV / HIV and HCV)

Application concentrations / application time

Application time as defined by Association of Applied Hygiene:

5 minutes

Application time as per expert assessment:

Prevention of hospital-borne infections	30 seconds
Bactericide under high load	30 seconds
Levurocide under high load	30 seconds
TB and Mycobacterium terrae	1 minute
Hepatitis C (BVDV)	30 seconds
HBV viricide (HIV) without protein load	1 minute
HBV viricide (HIV) with protein load	3 minutes
Adenovirus type 5 low load	5 minutes
Human rotavirus low load	1 minute
Polio virus type 1 EN 14476 + A1 low load	5 minutes

Product composition

The saturation solution contains:
Ethanol, 2-Propanol, N,N-Didecyld-N,N-dimethylammonium chloride, fragrance, pH regulators

Chemical and physical details

Appearance: clear, colourless
pH: approx. 5,0
Density (20°C): 0,900–0,920

Precautionary statements

- P210** Keep away from heat / sparks / open flames / hot surfaces - no smoking.
- P243** Take precautionary measures against static discharge.
- P280** Wear protective gloves / protective clothing / eye protection / face protection.
- P303 / P361 / P353** IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.

**P403 / P235
P501**

Labelling

CE 0029 / UN no. 3175

Storage conditions

Do not use after expiry date. After opening the disinfectant wipes refill bag, store the wipes in a container that can be closed again, such as a tin, in order to prevent them from drying out.

Special note

Always read the Instructions for Use and the Material Safety Data Sheet before use.

Keep out of reach of children.

Delivery units

Dispenser box/ Lemon	120 wipes	REF D0032
Refill bag / Lemon	120 wipes	REF D0033

Expert opinions: Available on request.

9999-D0032-03

Revision date 2015-03

IHDENT® and **Dentatrend®** are registered trademarks.

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de
www.desinfektion.com

Danger

H226
Flammable liquid and vapour.

H319
Causes serious eye irritation.

Legend

	Read instructions
	Expiration date
	LOT Charge number

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