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9999-D0044-03 V003

**Type of preparation**

IHIDENT® Dentatrend® Surface disinfection is a combined preparation for the disinfection and cleaning of surfaces and medical products. It is based on quaternary ammonium compounds and biguanides. IHIDENT® Dentatrend® Surface disinfection is free of aldehyde and phenol, and fulfils the legal occupational safety requirements laid down in TRGS 525/540.

**Areas of application**

Disinfecting and cleaning all kinds of surfaces on medical products, such as:

- Hospital beds
- Operating tables
- Treatment chairs
- Incubators (laboratories), etc.
- Floors, PVC / linoleum, tiles, etc.
- All areas of medicine, hospitals, surgeries and dental practices
- IHIDENT® Dentatrend® Surface disinfection is especially useful for areas in which, aside from fulfilling high hygienic standards, strong odours are to be avoided. Also suitable for acrylic glass!

**Microbiology**

IHIDENT® Dentatrend® Surface disinfection is effective against: Bacteria / Fungi (Candida albicans) / Bovine viral diarrhoea / Vaccinia virus

**Application concentrations / Application time**

Prepare the ready-to-use solution in the required concentration. Do not use the prepared solution for more than one day.

**Efficacies:**

DGHM list / certificate

Surface disinfection in hospitals and medical practices	1 % (10 ml/l)	5 min.
	0,75 % (7,5 ml/l)	15 min.
	0,5 % (5,0 ml/l)	30 min.

Limited viricide against BVDV (HCV surrogate virus)	0,5 % (5,0 ml/l)	5 min.
---	------------------	--------

Vacciniavirus		
Without load	0,5 % (5,0 ml/l)	5 min.
With load	1 % (10 ml/l)	30 min.

**Dispensing aids**

Dispensing pump for 5 l canisters  
Outlet tap for 5 l canisters  
1 l dispensing bottle

**Product composition**

Alkyldimethylbenzylammonium chloride, 2-Propanol, Coco propylenediamine-1.5-bis-guanidinium acetate, 1.3-Propylenediamine, N-(3-aminopropyl)-N-dodecyl

**Chemical and physical details**

Appearance of concentrate: green liquid  
pH: approx. 10  
Rel. density: 0,995 (± 0,010)

**Precautionary statements**

<b>P101</b>	If medical advice is needed, have product container or label at hand.
<b>P102</b>	Keep out of reach of children.
<b>P103</b>	Read label before use.
<b>P260</b>	Avoid breathing dust / fume / gas / mist / vapours / spray.
<b>P280</b>	Wear protective gloves / protective clothing / eye protection / face protection.
<b>P273</b>	Avoid release into the environment.
<b>P301 / P330 / P331</b>	IF SWALLOWED: rinse mouth. Do NOT indu-

ce vomiting.

**P303 / P361 / P353** IF ON SKIN (or hair): remove / take off immediately all contaminated clothing. Rinse skin with water / shower.

**P305 / P351 / P338** IF IN EYES: rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing.

**P304 / P340** IF INHALED: remove victim to fresh air and keep at rest in a position comfortable for breathing.

**P309 / P310** IF EXPOSED OR YOU FEEL UNWELL: immediately call a POISON CENTRE or doctor / physician.

**P501** Dispose of contents / container in compliance with local / regional / national / international regulations.

**Labelling**

CE 0029 / UN no. 1903

**Storage conditions**

Do not use after expiry date. Keep container tightly closed. Store container in a well-ventilated place. Keep locked up.

**Special note**

Always read the Instructions for Use and the Material Safety Data Sheet before use.

**Keep out of reach of children.****Delivery units**

5 L\* REF D0044

**Expert opinions:** Available on request.

9999-D0044-03

Revision date 2015-11

IHIDENT® and Dentatrend® are registered trademarks.

\* Only available via distributors.

**Dr. Ihde Dental AG**

Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

**Distribution by****Dr. Ihde Dental GmbH**

Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de  
www.desinfektion.com

**Danger****H314**

Causes severe skin burns and eye damage.

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