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IHDENTAL

Preparation type

IHDENT® Dentatrend® Impression Disinfectant is a submersive impression disinfectant with a new active ingredient which is characterised by short contact times and low use concentration. The gentle disinfectant and cleaning agent has a powerful and highly efficient cleaning action with a broad spectrum of efficacy. IHDENT® Dentatrend® Impression Disinfection contains corrosion stabilisers to protect the impression tray and contains no aldehyde or phenol. Its simple application allows cleaning and disinfection in one process.

Areas of use

IHDENT® Dentatrend® Impression Disinfectant is a concentrate permitting rapid disinfection and thorough cleaning of prosthetic materials such as impressions made of polyether, A/C silicones, alginates and hydrocolloids.

Use

With impression trays containing alginates, hydrocolloids and other materials for example, place for 15 minutes in a 6% solution and then dry for 10 minutes. The user must ensure that inchoate or empty containers are not refilled or filled with foreign liquids without the label being made unusable or removed.

IHDENT® Dentatrend® Impression Disinfectant as a prepared working solution should not exceed a life of 1 working day in order to guarantee full microbiological efficacy. In the case of more substantial contamination, the prepared solution should be renewed if required.

For hygiene reasons, impressions should be disinfected in the dental practice before being sent to dental laboratories in order to avoid microbial or infectious illnesses. If the dental laboratory receives an impression and is unsure whether disinfection has been performed, disinfection must at any rate be performed before work is started.

Important:

IHDENT® Dentatrend® Impression Disinfectant should be used in an instrument tray. The instrument tray must be cleaned thoroughly before the first use of IHDENT® Dentatrend® Impression Disinfectant. It must not contain residues of any other solutions.

Concentration for use / contact time

Bactericide: incl. Tbc and M. terrae	6% (60 ml/l)	15 minutes
Yeasticide: C. albicans	6% (60 ml/l)	15 minutes
Virucide:		
HBV / HIV	no contamination 2% (20 ml/l)	15 minutes
HBV / HIV	with contamination 4% (40 ml/l)	120 minutes
HCV	with contamination 1% (10 ml/l)	5 minutes

Dosing aids

Dosing pump for 5 litre canister
Tap for 5 litre canister
1 litre dosing bottle

Product composition

The concentrate contains:
2.50 g N,N-Bis (3-aminopropyl)dodecylamine
3.15 g didecyl-oxyethyl-methyl-ammonium propionate
2.00 g coco-propylene-diamine diguanidinoacetate

Chemical-physical data

Appearance of the concentrate: clear, pale blue liquid
pH value: approx. 7.2-8.00
Density (20 °C): 0.990-0.999

Safety instructions

P101	If medical advice is sought, take packaging or label with you.
P102	Must not be accessible to children.
P103	Read identification label before use.
P260	Do not inhale mist / vapour / aerosol.
P280	Wear protective gloves / protective clothing / protective goggles / face protection.
P273	Do not release into the environment.
P301 / P330 / P331	IF SWALLOWED: Rinse mouth. DO NOT induce vomiting.
P303 / P361 / P353	In case of contact with the skin (or hair): Immediately remove all contaminated clothing. Wash hair with water / shower.
P305 / P351 / P338	IN CASE OF CONTACT WITH THE EYES: Gently rinse with water for several minutes. Remove contact lenses if possible. Continue to rinse.
P304 / P340	IF INHALED: Get some fresh air and rest in a position that alleviates breathing.
P309 / P310	IN CASE OF exposure, immediately call POISON CENTRE or doctor.

Designation

CE 0029 / UN number 1903

Storage instructions

Do not continue to use after the use-by date. Protect from frost. Keep containers in a well-ventilated room. Store in a cool and dry place in tightly sealed containers. Comply with provisions under water law. Store in the original container.

Environmental information

Dr. Ihde Dental AG manufactures its products in accordance with advanced, safe and environmentally-friendly procedures, cost-effectively and in compliance with high quality standards (EN ISO 9001, EN ISO 13485). Biological degradability, which has been verified in expert reports, shows that disposal has a very low impact on residual water and the environment and is thus environmentally friendly.

Environmental indications

Biological degradability: slightly biologically degradable
Method: OECD 301D / EEC 84/449 C6

Special notes

Always read the instructions for use and the safety data sheet prior to use.

Keep out of reach of children.

Package sizes

2 L REF D0210
5 L* REF D0211

Expert report: Available; please request if you would like a copy.

9999-D0210-03

Last revised 2015-09

IHDENT® and **Dentatrend®** are registered trademarks.

* Only available from distributors.

Dr. Ihde Dental AG
Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

**Distribution by
Dr. Ihde Dental GmbH**
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de
www.desinfektion.com

Hazards

H314
Causes severe skin burns and serious eye damage.

H400
Highly toxic to water-borne organisms.

Legend

	Read instructions
	Expiration date
	Batch number

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