Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® Allfit® Pro v

Essential components

Allfit® Pro v is a dual-curing, semi-permanent luting cement based on a multifunctional methacrylate.

Allfit® Pro v is used for luting implant-supported crowns and bridges. The material possesses good elastic properties and produces an excellent seal due to its controlled minimal expansion. Allfit® Pro v does not affect the setting of composite cements and composites used at a later stage.

Application

Preparing the syringe

Remove the seal of syringe (discard, do not reuse!) and replace with a 4:1 mixing tip. Apply the material directly from the mixing tip.

Note

The used mixing tip should remain fitted as a seal until the next application. The first 2 - 3 mm of material extruded from the mixing tip should be discarded. This applies to every new mixture.

Application

The material is applied directly from the mixing tip into the restoration using light pressure. Apply a very small quantity – enough to produce a thin film – of the mixed material to the inside of the restoration.

Then fit the restoration carefully to the implant after a max. of 2 min. (working time at 23 °C) using light pressure initially followed by firmer pressure.

Allfit® Pro v cures intraorally within approx. 4 min. Allfit® Pro v can also be light cured for 20 seconds to ensure quick retention of the restoration. Remove the excess after curing. Remove any smear layer that may have formed with alcohol.

Precautions

- Avoid contact with the skin, particularly in the case of people with a known allergy to methacrylate.
- On contact with the skin and eyes, wash immediately with plenty of water.
- Consult a doctor immediately if swallowed.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Keep out of reach of children.

Intended only for use by dentists.

Storage

Do not store over 20 °C. Do not use after the expiry date.

Packaging

Supplied in 4:1 syringes. Allfit® Pro v is very easy and convenient to use.

Package size

Double syringe with 5 ml, 5 mixing cannulas

Shade: universal **A3**

REF 2135

Mixing cannulas à 25 pieces

REF 2129

Legend

Read instructions

Expiration date

LOT Charge number

Store protected against the sun

Keep in a dry place

Store tightly closed keep

Temperatures up to 20 °C

Manufacturer

CE 0029

Warning

H315

Causes skin irritation.

H319

Causes serious eye irritation.

9999-2135-03

Revision issued

2015-07

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by Dr. Ihde Dental GmbH

Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT® and Allfit® are registered trademarks.

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