Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® E&B® 1

IHDENT® E&B® 1 is a simple to use self-etching light curing one component adhesive. It is designed for strong bonding of composites to enamel and dentin. IHDENT® E&B® 1 is compatible with all current brands of visible light cure composite restorative materials.

Essential components

IHDENT® E&B® 1 contains modified dimethacrylates and methacrylates, phosphoric acid esters, catalysts and stabilizers.

Application areas

- Adhesive for direct, light curing composite and compomer restorations.
- Adhesive for indirect full-ceramic and composite restorations (veneers, inlays, onlays; only for light-curing applications).

Contraindication

IHDENT® E&B® 1 if a patient is known to be allergic to any of the materials ingredients or if the stipulated working technique cannot be used. IHDENT® E&B® 1 is a light-curing adhesive only and should not be used in conjunction with self-curing resin materials. Dual-curing materials must be light initiated.

Side effects

IHDENT® E&B® 1 may cause a sensitizing reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be used.

Interaction

Materials containing eugenol or clove oil may inhibit the polymerization of IHDENT® E&B® 1.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of hard setting calciumhydroxide material.

4. Application of IHDENT® E&B® 1

Apply IHDENT® E&B® 1 with a brush onto the moist enamel and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 20 seconds before placement of a composite.

5. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer. Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P262 Do not get in eyes, on skin, or on clothing.

Storage

Do not store above 25 °C (78 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

Keep away from children!
For dental use only!

Package IHDENT® E&B® 1

Bottle with 5 ml

REF 5140

Legends

Read instructions

Expiration date

LOT Charge number

Store protected against the sun

Keep in a dry place

Store tightly closed keep

Temperatures up to 25 °C

Manufacturer

CE 0029

Warning

H315

Causes skin irritation.

H317

May cause an allergic skin reaction.

H319

Causes serious eye irritation.

9999-5140-03

Revision issued 2015-07

Dr. Ihde Dental AG

Dorfplatz 11

CH-8737 Gommiswald/SG

Tel. +41 (0)55 293 23 23

Fax +41 (0)55 293 23 00

contact@implant.com

www.implant.com

Distributed by

Dr. Ihde Dental GmbH

Erfurter Str. 19

D-85386 Eching/München

Tel. +49 (0)89 319 761-0

Fax +49 (0)89 319 761-33

info@ihde-dental.de

www.ihde-dental.de

IHDENT® and E&B® are registered trademarks.

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