Deutsch:

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English:

Instructions for use – Please read carefully

## IHDENT® E&B® SD 1

IHDENT® E&B® SD 1 is a simple to use self-etching light curing one component adhesive. It is designed for strong bonding of composites to enamel and dentin. IHDENT® E&B® SD 1 is compatible with all current brands of visible light cure composite restorative materials.

### Essential components

IHDENT® E&B® SD 1 contains modified dimethacrylates and methacrylates, phosphoric acid esters, catalysts and stabilizers in ethanol.

### Application areas

- Adhesive for direct, light curing composite and compomer restorations
- Adhesive for indirect full-ceramic and composite restorations (veneers, inlays, onlays; only for light-curing applications)

### Contraindication

IHDENT® E&B® SD 1 if a patient is known to be allergic to any of the materials ingredients or if the stipulated working technique cannot be used. IHDENT® E&B® SD 1 is a light-curing adhesive only and should not be used in conjunction with self-curing resin materials. Dual-curing materials must be light initiated.

### Side effects

IHDENT® E&B® SD 1 may cause a sensitizing reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be used.

### Interaction

Materials containing eugenol or clove oil may inhibit the polymerization of IHDENT® E&B® SD 1.

### Application

#### 1. Isolation

Rubber dam is the recommended method of isolation.

#### 2. Cavity Preparation

Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

#### 3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of hard setting calciumhydroxide material.

#### 4. Application of IHDENT® E&B® SD 1

Pierce the supplied microtip vertical through the aluminum foil. Apply IHDENT® E&B® SD 1 with a brush onto the moist enamel- and dentin surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 20 seconds before placement of a composite.

#### 5. Restorative Placement

Apply the restorative material according to the instructions of the manufacturer. Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

### Precautionary statements

#### P305+P351+P338

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

#### P262

Do not get in eyes, on skin, or on clothing.

### Storage

Do not store above 25 °C (78 °F)! Avoid storage in direct sunlight. Do not use after expiration date. IHDENT® E&B® SD 1 in SingleDose is for single use only.

**Keep away from children!  
For dental use only!**

### Package E&B® SD 1

48 Tabs of 0,1 ml  
50 microtips with applicator

REF 5142

### Legends

Read instructions

Expiration date

LOT Charge number

Store protected against the sun

Keep in a dry place

Store tightly closed keep

Temperatures up to 25 °C

Manufacturer

## CE 0029

### Warning

#### H315

Causes skin irritation.

#### H317

May cause an allergic skin reaction.

#### H319

Causes serious eye irritation.

9999-5142-03

Revision issued 2015-07

### Dr. Ihde Dental AG

Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

### Distributed by

### Dr. Ihde Dental GmbH

Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de

IHDENT® and E&B® are registered trademarks.

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**Dr. Inhe Dental AG**