Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® E&B® SD 2

IHDENT® E&B® SD 2 is a simple to use self-etching dual-cure adhesive. It is designed for strong bonding of composites, compomers and resin modified glass ionomer cements to enamel and dentin.

IHDENT® E&B® SD 2 of two components in a SingleMix unit that were mixed before application. It is bonding to dual-cure, self-cure and light cure composite restorative materials.

Essential components

IHDENT® E&B® SD 2 contains dissolution of modified dimethacrylates, methacrylates, phosphoric acid esters, catalysts and stabilizers.

Indication

IHDENT® E&B® SD 2 is used for light-curing and chemically curing (dual-curing) composites, compomers and resin-reinforced glassionomer cements.

Contraindication

Do not use IHDENT® E&B® SD 2 if a patient is known to be allergic to any of the materials ingredients or if the stipulated working technique cannot be used. IHDENT® E&B® SD 2 is a light-curing adhesive only and should not be used in conjunction with self-curing resin materials. Dual-curing materials must be light initiated.

Side effects

IHDENT® E&B® SD 2 may cause a sensitizing reaction in patients with a hypersensitivity to any of its ingredients. In these cases, the material should no longer be used.

Interaction

Materials containing eugenol or clove oil may inhibit the polymerization of IHDENT® E&B® SD 2.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

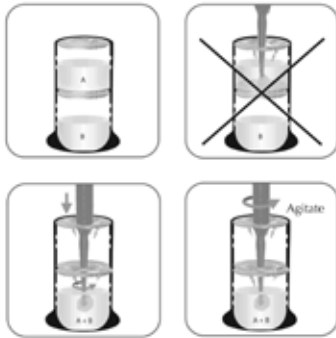
Clean the tooth with flour of pumice and water prior to preparation. Prepare the cavity with minimal tooth reduction. Margins should have a slight (0.5 - 1.0 mm) bevel placed in the enamel to increase the surface area for greater bond strength.

3. Pulp Protection

Cavity floor of deep excavations should be covered with a thin layer of calciumhydroxide material.

4. Application of IHDENT® E&B® SD 2

Pierce the supplied microbrush vertical through the aluminium foils of the Single-Mix unit up to the riffles of the microbrush. Ensure that both aluminium foils are penetrated completely. Mix the two components for 10 seconds.

Apply the homogeneous mixture generously with the microbrush onto the slightly wet enamel- and dentine surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure for 20 seconds with a dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) and place the restorative material. For a maximal adhesion it is strongly recommended to use the light cure mode. If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application of the adhesive air thin gently to remove all volatile components. There must remain a sticky layer. Then apply the restorative material directly. Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

5. Restorative Placement, Cure and Finishing

Refer to manufacturer instructions for placement, curing and finishing of the used restorative materials.

Precautionary statements

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P262 Do not get in eyes, on skin, or on clothing.

Storage

Do not store above 25 °C (78 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

IHDENT® E&B® SD 2 in SingleMix is for single use only.

Keep away from children!
For dental use only!

Pack IHDENT® E&B® SD 2 REF 5144
35 Tabs of 2 x 0,1 ml, 35 Microbrushes (green)

Legends

	Read instructions
	Expiration date
	LOT Charge number
	Store protected against the sun
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

Danger

H225
Highly flammable liquid and vapour.

H315
Causes skin irritation.

H317
May cause an allergic skin reaction.

H319
Causes serious eye irritation.

H335
May cause respiratory irritation.

9999-5144-04

Revision issued 2015-07

Dr. Ihde Dental AG
Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH
Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT® and E&B® are registered trademarks.

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