Deutsch:

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English:

Instructions for use – Please read carefully

## IHDENT® Ceramolar® Flow / IHDENT® Ceramolar®

Ceramolar® Flow / Ceramolar® are visible light cure Nano composite for anterior and posterior restorations. They are characterized by low shrink, low abrasion and an improved handling features.

### Essential components

Resins on the basis of urethane dimethacrylate, radio-opakes glass-filling, silicium dioxide, polymer powder, catalysts, stabilizers, pigments.

### Indications

Suitable for all cavities.

### Application

#### Preparation of tooth

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate Ceramolar® Flow / Ceramolar® shade.

Ceramolar® Flow / Ceramolar® are shaded according to VITA®-Shades. It is also available in special shades and bleach shades.

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. Best pulp protection in case of deep excavations is the photo-activated disinfection (PAD). If this is not possible, deep excavations should be covered with a thin layer of hard setting calciumhydroxide material.

Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with Etching Gel (e.g. IHDENT® Etching Gel, REF 5138) and subsequent application of a light cure bonding agent (e.g. IHDENT® P&B®, REF 5120) is recommended. Otherwise use a self etching bonding agent (e.g. IHDENT® E&B® 2, REF 5143) before application of the Ceramolar® Flow / Ceramolar®.

### Curing

Ceramolar® Flow / Ceramolar® in the selected shade. Ceramolar® Flow / Ceramolar® are easy to model. Place it by using a non-metallic instrument. Transparent matrix strips may be used.

In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure flowable composite, followed by the application of Ceramolar® Flow / Ceramolar®.

In case of small cavities Ceramolar® Flow / Ceramolar® can be applied directly. For an optimum result apply Ceramolar® Flow / Ceramolar® in layers of max. 2 mm. Light cure each increment separately.

By using a polymerization unit with a light intensity of at least 1000mW/cm², cure each increment as follows:

Lighter shades (e.g. A1, A2, C2) **20 seconds**

Darker shades (e.g. A3.5, B3) **30 seconds**

### Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

### Adverse reactions

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of hard setting calciumhydroxide material.

### Side effects

Do not use in combination with substances containing eugenol because eugenol can impair the hardening of the composite and causes discoloration. Do neither store the composite material in proximity of eugenol containing products, nor let the composite allow to come into contact with materials containing eugenol.

### Warning

APPLIC.-Tips are for single use only.

### Additional notes

Do not use any resin to adjust viscosity of composite restorative material. Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.

### For professional dental use only.

Keep away from children.

### Storage

Do not store above 25 °C (78 °F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

### Presentation

#### Ceramolar® Flow

Shades **Syringe 5 g, 3 applicator tips**

REF

Universal A2 4047

Universal A3 4048

Brown A3,5 4049

#### Ceramolar®

Shades **Syringe 4.5 g**

REF

Universal A2 4060

Brown A3,5 4064

Universal A3 4065

Shades **20 Tips per 0,3 g**

REF

Universal A2 4070

Brown A3,5 4074

Universal A3 4075

### Legends

Read instructions

Expiration date

LOT Charge number

Store protected against the sun

Keep in a dry place

Store tightly closed keep

Temperatures up to 25 °C

Manufacturer

CE 0029

DIN EN ISO 4049:2009

9999-4047-03

Revision issued 2015-08

**Dr. Ihde Dental AG**

Dorfplatz 11

CH-8737 Gommiswald/SG

Tel. +41 (0)55 293 23 23

Fax +41 (0)55 293 23 00

contact@implantf.com

www.implantf.com

**Distribution by Dr. Ihde Dental GmbH**

Erfurter Str. 19

D-85386 Eching/München

Tel. +49 (0)89 319 761-0

Fax +49 (0)89 319 761-33

info@ihde-dental.de

www.ihde-dental.de

IHDENT® and Ceramolar® are registered trademarks.

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Vita® is a registered trademark of Vita Zahnfabrik H. Rauter GmbH.