Deutsch:

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English:

Instructions for use – Please read carefully

## IHDENT® **Experdent® APH**

Light curing composite restorative material for all cavity classes. The total inorganic filler content is 75 wt. % (0.02-15 µm).  
Total filler content: 83 wt. % (75 Vol. %)

### Essential components

Contains dental glass in a Bis-GMA based matrix of dental resins and polymer fillers. Additionally, small amounts of silicon dioxide, pigments, additives and catalyst.

### Indications

- Fillings for cavities (Class I - V)
- Aesthetical corrections
- Morphological corrections in the area of the enamel
- Repair of ceramic and plastic veneers
- Core buildups

### Application

1. Clean the tooth before preparation, remove all residues with water spray. Choose Vita-shade.
2. Avoid saliva contamination during application.
3. Prepare the cavity according to the general rules for adhesive dentistry. Bevel enamel margins. Protect pulp in deep cavities.
4. Etch enamel margins with IHDENT® Etching Gel (REF 5138) according to it's instructions.
5. Rinse off etching gel completely.
6. Dry with oil-free air. Avoid dentin desiccation.
7. Apply the adhesive (IHDENT® P&B®, REF 5120) according to its instructions.
8. Apply Experdent® APH in layers of 2 mm max.
9. Light cure each layer for 40 seconds.
10. Repeat steps 8 and 9, until the cavity is filled.
11. Remove excesses, shape and finish with diamond burs and flexible disks of increasing grits. Depth of cure (40 seconds): 3.5 mm (opa-que shades 3.0 mm).

### Adverse reactions

Side effects are not known to date. In singular cases allergic reactions of type IV have been described.

### Contraindications

Application of Experdent® APH is contraindicated, when a dry working area is not possible, for direct application on the opened pulp, when allergies against components of Experdent® APH are known.

### Side effects

Avoid materials containing eugenol, moisture and oily air; they can inhibit the polymerization at the contact area. The use of cationic mouth washes, plaque indicators and chlorohexidine may lead to discolourations.

### Storage

Store at room temperature (15 - 25 °C / 59 - 77 °F). Please close syringes immediately after use to avoid premature settings owing to ambient light. Do not use after expiry date. Lot No and expiry date see label on syringe or barcode on the package.

### Essential notes

Light curing units should have 450 nm output and should be checked regularly. Place the light as close as possible to the restorative material. Do not disturb the oxygen inhibited layer because it is needed for bonding to the next layer. The use of a matrix is advised. Follow the instructions for use of any materials which are used with Experdent® APH.

### Precautionary statements

- P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P302+P352** IF ON SKIN: Wash with plenty of soap and water.
- P280** Wear protective gloves / protective clothing / eye protection / face protection.

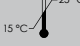
**Keep away from children. For dental use only.**

### Packaging units

1 syringes 3 g paste in the shades

Universal A2	REF 4013
Light B1	REF 4014
Yellow B3	REF 4015
Brown A3,5	REF 4017
Universal A3	REF 4018

### Legends

	Read instructions
	Expiration date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperature range from 15 °C to 25 °C
	Manufacturer

CE 0029

DIN EN ISO 4049

### Warning

- H315**  
Causes skin irritation.
- H317**  
May cause an allergic skin reaction.
- H319**  
Causes serious eye irritation.
- H335**  
May cause respiratory irritation.

9999-4013-03

Revision issued 2015-08

**Dr. Ihde Dental AG**  
Dorfplatz 11  
CH-8737 Gommiswald/SG  
Tel. +41 (0)55 293 23 23  
Fax +41 (0)55 293 23 00  
contact@implant.com  
www.implant.com

### Distribution by

**Dr. Ihde Dental GmbH**  
Erfurter Str. 19  
D-85386 Eching/München  
Tel. +49 (0)89 319 761-0  
Fax +49 (0)89 319 761-33  
info@ihde-dental.de  
www.ihde-dental.de

IHDENT®, Experdent® and P&B® are registered trademarks.

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Vita® is a registered trademark of Vita Zahnfabrik H. Rauter GmbH.