Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® **CemBase®** blue (Powder)

CemBase® is a water-mixable glass-ionomer cement used for cavity liners, cementation, core build-ups under crowns and bridges as well as orthodontic dentistry. CemBase® blue has an intense blue self colour. This enables the operator to distinguish the material easily from the tooth structure. This is particularly practical when it is used for core build-ups and cavity liners or cementation of bands (as well as removal of bands).

The natural blue dye contained in CemBase® is food safe, nontoxic and does not penetrate the dental enamel or dentine.

CemBase® blue is self curing and water mixable.

Essential components

Calcium fluoride, silicate, aluminium oxide, aluminium phosphate, aluminium fluoride, sodium fluoride, dried polyacrylic acid.

Application

CemBase® blue is mixed with normal tap water. It can also be used with distilled water. The application surfaces should be thoroughly cleaned to ensure an optimal bond to enamel, dentine and metal. The dentine and enamel can also be conditioned with IHDENT® GIZ® Conditioner (REF 2330).

Pulp protection is recommended in the case of deep cavities; CemBase® should not be used for direct pulp capping.

Dispensing

Shake the powder thoroughly before use. Mix 2 scoops of powder with 2 drops of water quickly and intimately. The powder and water can be mixed on a cooled glass plate to delay setting.

Increasing the proportion of powder will generally produce more stable curing of CemBase® blue. It should, however, be taken into consideration during cementation that a higher viscosity of cement also produces greater pressure in crowns and bridges. This may make it difficult or impossible to position the restoration correctly on the prepared tooth.

A metal spatula can be used for mixing. We recommend mixing 2 - 3 small portions of powder into the liquid at a time to attain better wetting of the powder.

Working time

The following times (in minutes) apply for the individual working stages at a room temperature of 23 °C:

- 00.00** Beginning of mixing
- 00.30** End of mixing
- 02.00** End of working time
- 05.00** Final set

Higher temperatures shorten the setting time and lower temperatures extend the setting time.

Exceeding the working time will result in a significant loss of adhesion to enamel and dentine.

Application

Apply a uniform film of cement to the prepared tooth and the inside of the crown. Apply a coat of cement to the inside of bands when bonding in orthodontics. It is helpful to use a higher viscosity when cementing bands.

Excess cement should only be removed once the cement has fully set. The cement can be protected against water penetration, if necessary, using a light-curing bonder or fissure sealant.

Note:

1. CemBase® blue adheres very easily to metal instruments and should be completely removed from the instruments before setting.
2. The powder should be stored tightly sealed, cool and very dry.
3. CemBase® blue can be used up to three years after the date of manufacture. After this time a reduction in adhesion is to be expected.
4. Avoid contact of mixed CemBase® blue with unprotected skin or eyes. If there is contact, remove immediately with plenty of water.

Side effects / Interactions

Severe irritation of the pulp may occur with any glass ionomer cement. It cannot be ruled out that CemBase® blue may cause sensitisation in sensitive patients. CemBase® blue should no longer be used if these symptoms occur. CemBase® should not be used if there is a suspected allergy. If it is necessary to remove the CemBase® blue in such cases, a rubber dam should be placed before removal to prevent components of the material infiltrating the oral mucosa.

Storage

The maximum storage temperature is 25 °C. Store dry, tightly close the container after use. Refer to the barcode for batch number and expiry date.

Intended only for use by dentists. Keep out of reach of children.

Pack size

Pack with 50 g powder, accessories

REF 8521

Legend	
	Read instructions
	Expiry date
	LOT Charge number
	Keep in a dry place
	Store tightly closed keep
	Temperatures up to 25 °C
	Manufacturer

CE 0029

ISO 9917:2007

9999-8521-03

Revision issued 2015-08

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH

Erfurter Str. 19
D-85386 Eching/München
Tel. +49 (0)89 319 761-0
Fax +49 (0)89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT®, CemBase® and GIZ® are registered trademarks.

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