Deutsch:

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English:

Instructions for use – Please read carefully

## IHDENT® SIL 18

SIL 18 is a thin viscosity duplicating material on the basis of addition-curing silicones with a mixing ratio of 1:1.

It is ideal for demanding model casting and combination work. SIL 18 is characterised by its high precision. It is practically shrinkage-free, tear-resistant and can be manually mixed in a dosing machine. A very long storage time of material and duplicating mould can be guaranteed. Multiple pouring is possible.

### Essential components

Based on polyorganosiloxanes and inactive, inorganic fillers. Contains 7% polymethylhydrogensiloxane.

### Characteristics

Appearance of liquid  
Colour

component A white,  
component B green

Mixing time approx. 30 seconds  
Working time at 23°C approx 6 minutes  
Setting time at 23 °C approx. 20 minutes  
Removal time approx. 25 minutes  
Shore hardness A after 30 minutes approx. 17  
Shore hardness after 24 hours approx. 18

### Application areas

Duplication of plaster models

### Application

Add 100 parts SIL 18 component A with 100 parts SIL 18 component B. Mix together thoroughly in a clean container until a uniform colour is achieved; preferably using a vacuum mixing device. Pour over the original model. Remove only after at least 30 minutes.

### Note

Mixing may only be carried out in a clean stainless steel or plastic container. Polyaddition crosslinking reactions can be inhibited by contact with substances such as:

- heavy metal salts
- amines
- sulphur and sulphur derivatives
- catalysts of epoxy resins

### Important note

The information and data contained in the directions for use are based on our current state of knowledge of the product. As the application, use and preparation of the products are not able to be controlled by Dr. Ihde Dental, we do not assume any guarantee for their suitability for a purpose other than that specified. The application recommendations are not to be construed as inducements to infringe any relevant patents.

### Warnings

Please observe the safety data sheet for SIL 18 components A and B. Higher temperatures shorten times and lower ones increase times.

### Storage

SIL 18 component A = white and B = green can be stored in unopened packages at a temperature of less than 30 °C for 12 months.

### Package sizes

SIL 18 component A = white and B = green are available in the following package sizes.

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IHDENT® is a registered trademark.
**Dr. Ihde Dental AG**