Deutsch:

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English:

Instructions for use – Please read carefully

IHDENT® Maxit®

Maxit® is a temporary filling material for dental cavities.

Constituents

Composition of calcium sulfate, zinc oxide, butyl phthalate plasticizer

Therapeutic indications

Maxit® is suitable for the temporary closure of all types of dental cavities and for the temporary closure of root canals as well as the blocking of model cavities in the dental laboratory.

The maximum length of use is considered to be 3 - 4 weeks in healthy teeth.

Use

Maxit® is to be removed from the glass bottle with a clean spatula and immediately introduced into the prepared cavity with moderate pressure. Maxit® is thoroughly adapted to the cavity edges with a round condenser or a Heidmann spatula. For a secure closure, the minimal layer thickness should not be less than 1.5 mm. Deep cavities (i.e. in the area of canal entrance) can be filled with cotton in the basal portion since Maxit® can be difficult to remove in very deep regions.

Maxit cures within 3 minutes, as soon as it comes in contact with saliva, and can be burdened with normal chewing pressure after 15 minutes.

Removal

Maxit® can be removed from the cavity with a tartar remover or with a probe.

Note: Material which has been removed may not be returned to the glass bottle!

Contraindications / Interactions

None known when used accordingly. Maxit® should not be used for closure over toxic medicational inserts. Further information can be found in the safety information sheet.

Side effects

If Maxit® is introduced into a very dry cavity, a short term pain may be present due to water absorption from the dentin tubules.

Storage / Shelf life

Store the glass bottles tightly closed in a cool, dark place. The storage in damp rooms causes a premature curing of the material (also see the safety data sheet). Do not use after the expiration date. For charge number and expiration date see the bar code on the packaging or label on the bottle.

Note

Only for dental use. Store out of reach of children.

Package sizes

2 glass bottles of 30g REF 2090

Color: white

Legends

Read instructions

Expiration date

LOT Charge number

Store protected against the sun

Keep in a dry place

Store tightly closed keep

Temperatures up to 22 °C

Manufacturer

CE 0029

9999-2090-03

Revision issued 2015-08

Dr. Ihde Dental AG

Dorfplatz 11
CH-8737 Gommiswald/SG
Tel. +41 (0)55 293 23 23
Fax +41 (0)55 293 23 00
contact@implant.com
www.implant.com

Distribution by

Dr. Ihde Dental GmbH

Erfurter Str. 19
D-85386 Eching/München
Tel. +49 89 319 761-0
Fax +49 89 319 761-33
info@ihde-dental.de
www.ihde-dental.de

IHDENT® and Maxit® are registered trademarks.

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